

# KeyPrint® Post Cure Units

Status: June 24, 2019

	Model	KeyModel®	KeyOrthoModel®	KeyCast®	KeyMask™	KeyGuide™	KeySplint Soft™
OtoFlash	G171	4000 flashes per side	4000 flashes per side	2000 flashes per side	4000 flashes per side	5000 flashes per side	2000 flashes per side
Uvitron	Intelliray	2 minutes per side	2 minutes per side	1 minute per side	2 minutes per side	2 minutes per side	2 minutes per side
Evodent		10 minutes in water	10 minutes in water	10 minutes in water	10 minutes in water	pending	pending
NextDent (3D Systems)	LC-3DPrint Box	30 minutes per side	30 minutes per side	15 minutes per side	30 minutes per side	30 minutes per side	pending
Dymax	5000-EC	4 minutes per side	4 minutes per side	1 minute per side	4 minutes per side	pending	pending
Kulzer	Hilite	pending	pending	pending	pending	pending	pending
Dreve	FotoDent Flash	pending	pending	pending	pending	pending	pending
Structo	PCU	pending	pending	pending	pending	pending	pending