



KEYMODEL ULTRA™

# Phrozen Sonic XL 4k Printer Instructions



KeyModel Ultra™ Light Grey 50µm

## PRINTER SETTINGS

PRINTER SETTINGS	KeyModel Ultra™ Light Grey 50µm
Thickness (mm)	0.05
Down Speed	Not to exceed 250
Delay (ms)	1-2 seconds
Burn in Layers	6
Burn in Exposure	35000
Burn in Height	8-10
Burn in Up Speed	200
Normal Exposure	3500
Normal Height	7-10
Normal Up Speed	60-80